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Medical device of class 1 according to MDR (EU) 2017/745.

This device is intended to be handled by anyone able to read and understand the instructions for use.

Patient profile: Any type patient whose pathology requires the use a device to prevent pressure sores and/or treatment aid.

Operator's profile for installation, implementation, adjustments and maintenance: Patient, healthcare professionals, maintenance technicians, third party. There are no particular restrictions on the handling and use of the device.

CHECKS TO BE CARRIED OUT

At each cleaning and disinfection, at the start and end of treatment.

-Interior and exterior cleanliness
-Inventory and location of any damage. Change if: cuts, rips, cracks or tears, broken seams or welds, or defective zip.

In case of damaged device (foam deformation or foam collapse, loss of hardness leading to bottoming effect, leakage), or of persistent odor, replace the medical device.

CLEANING - DISINFECTION - RECONDITIONING

Information for healthcare professional

⚠ Medical devices require rigorous hygiene practice. Gloves and eyes protection are recommended for the entire cleaning and disinfection procedure.

Cleaning, disinfection and reconditioning must be carried out in accordance with the guidelines and recommendations of the health and hygiene authorities.

When repackaging the support, make sure that the instructions for use for the next installation are included. It is recommended to keep an inspection and maintenance tracking of the device.

Cleaning & disinfecting of covers

Frequency:

-At each change of bed sheets
-In case of contamination
-After each patient change

Cleaning: Surface cleaning with a wipe / sponge lightly moistened with soapy water and then rinsing with a wipe / sponge lightly moistened with clear water.

Disinfection: Surface disinfection with a wipe / sponge lightly moistened with a pH-neutral surface disinfecting detergent adapted to medical devices (in conformity with the procedure recommended by the manufacturer of the used product). Proceed with a final rinsing with a wipe / sponge lightly moistened with clear water. Dry thoroughly. Do not use corrosive products or abrasive materials.

Cleaning in a wash machine of the cover

-After every 20 sprays of disinfecting detergent
-After each patient change

Jersey cover (solely for single patient-use)

Machine wash at 65°C. Tumble-dry at moderate temperature. Do not iron. Do not dry clean. Do not use a corrosive product or an abrasive material.

POLYFILM® cover (solely for single patient-use)

Hand wash in antiseptic soapy water, rinse in clear water. Dry in open air. Do not use bleach. Do not iron. Do not use a tumble dryer. Do not iron. Do not use corrosive products or abrasive materials.

POLYMAILLE® and POLYMAILLE®HD covers:

Machine wash at 95°C. Maximum allowed chlorine concentration < 1000 ppm (in this case, rinsing with clear water mandatory). Tumble-dry at moderate temperature. Do not iron. Do not dry clean. Do not use a corrosive product or an abrasive material.

Precautions to follow:

-**Always let the cover dry thoroughly** before using it to cover the device (risk of mould if the device covered with a moist or poorly dried cover).
-Regularly check (at each servicing time) the condition of the cover for tears, perforations, signs of abrasion, and stop using the device if the cover has a problem of integrity.
-Use an identical spare cover if the device is used during the cleaning of the cover.

Cleaning & disinfection of device

Frequency:

-Each time the cover is removed for washing
-In case of damage on the cover
-After each patient change

Cleaning: Surface cleaning with a wipe / sponge lightly moistened with soapy water and then rinsing with a wipe / sponge lightly moistened with clear water.

Disinfection: Surface disinfection with a wipe / sponge lightly moistened with a pH-neutral surface disinfecting detergent adapted to medical devices (in conformity with the procedure recommended by the manufacturer of the used product). Proceed with a final rinsing with a wipe / sponge lightly moistened with clear water. Dry thoroughly. Do not use corrosive products or abrasive materials.

Precautions to follow:

-Do not immerse the foam
-Allow the device to dry in the open air before refitting the cover to the equipment (risk of mould forming if the core is covered with a wet or incompletely dried cover)
-Do not use corrosive products or abrasive material.
-Do not use or reuse a contaminated or uncleaned device.

List of products recommended / prohibited

Recommended disinfecting products (non-exhaustive list):

Cold disinfecting sprays containing quaternary ammoniums, amino-acids, glutaraldehydes, biguanides, propionate of didecylmethyl, alkylamine, ethanol and isopropanol (below 30% alcohol concentration), solution containing chlorine (concentration below 1000 ppm, rinsing with clear water mandatory).

Prohibited products:

Do not use solutions containing: phenol, inorganic acids at high concentration, cresol, caustic soda.

GUARANTEE

Two years against any manufacturing defaults, from the date of purchase by the customer. Small imperfections (small tears, marbling, surface defects, etc.) due to the manufacturing process in no way affect the quality and effectiveness of the product and cannot be a reason for replacing the product under warranty. The warranty does not cover misuse of the support, normal wear of the device or of its cover. In the event of default, please contact your provider, and provide the product lot number.

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