
ROHO®

LTV Seat® Cushion

Owner's Manual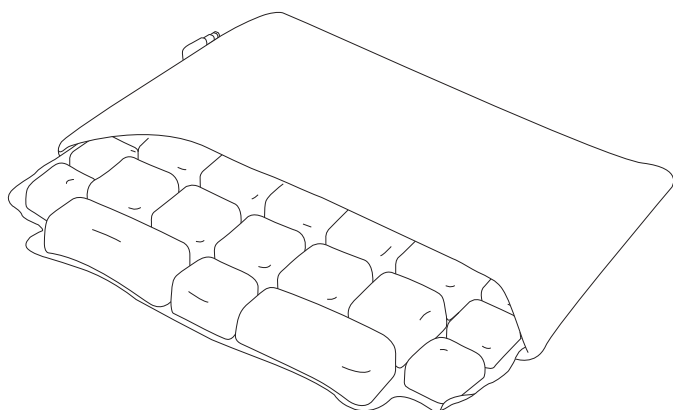


permobil

Package contents: LTV cushion, LTV cover, repair kit, hand pump, owner's manual

⚠ Warnings: DO NOT use the cushion as a water flotation device (e.g., a Life Preserver). It will NOT support the body in water. Keep the cushion away from sharp objects and hot ashes.

To Adjust the LTV Seat:

1. Open the valve. Inflate with the hand pump until all of the air cells are visibly filled. Close the valve and then remove the hand pump. Make sure the inflation valve is closed. *Note: To open, turn the inflation valve at least one full rotation.*
2. Place the LTV Seat on the sitting surface:
The non-skid side of the LTV Cover should face down, and the zippered portion of the cover should face the back. The valve should be on the front left corner.
3. Sit on the LTV Seat in normal sitting position.
4. Open the valve to release air. As air is released, slowly sink into the LTV Seat. Release air until close to the desired comfort level, and then close the valve. Release air or add more air to reach the desired comfort level.

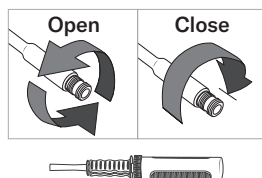To clean the LTV Cover:

Remove the cover from the cushion. If the cover is Quilted Fabric: Machine wash cold, gentle cycle, with like colors. Air dry.

To clean the LTV Cushion:

Close the valve and place the cushion in a large basin. Use a liquid dishwashing detergent, laundry detergent, or another multipurpose detergent that is used for general cleaning. Follow the surface-cleaning directions on the product label. Use a soft plastic bristle brush, sponge, or cloth to gently scrub all surfaces. Rinse the cushion with clean water and air dry or towel dry.

⚠ Cautions: DO NOT expose to direct sunlight. • DO NOT use automatic dishwasher detergents, which are too caustic for personal safety.

Note: Deflated air cells may be easier to thoroughly clean. To deflate the air cells: with the inflation valve open, roll up and/or compress the cushion, and then completely close the inflation valve.

Warranty: This product is covered by a 1 year warranty against manufacturer defects with the exception of product covers that are eligible for warranty up to 6 months. If a manufacturing defect should occur, discontinue use immediately. Wear from regular use, misuse, damage by an accident or damage from "an act of Nature" (e.g., flood, tornado, earthquake and/or fire) is not considered a manufacturing defect. Replacement parts are available by contacting the supplier, distributor, or retailer. For warranty details, please visit permobil.com or contact Customer Support to request a printed copy of the full warranty policy.

 ROHO, Inc., 1501 South 74th St.
Belleleville, Illinois 62223 U.S.A.
U.S.A.: 1-800-736-0925 • Fax: 888-551-3449
Customer Support: orders.roho@permobil.com
Outside the U.S.A.: +1 618-277-9150
Fax: +1 618-277-6518
intlorders.roho@permobil.com
permobil.com
© 2005, 2024 Permobil
Rev. 2024-12-16

LTV Cushion made in the U.S.A.
Products may be covered by one or more U.S. and foreign patents and trademarks, including LTV®, LTV Seat®, ROHO®, and shape fitting technology®.
Product information is changed as needed; current product information is available at permobil.com.
ROHO, Inc. is a part of Permobil.

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Bellefonte, Illinois 62223 U.S.A.
ROHO, Inc. é parte do Permobil.